



VisuALS

We at VisuALS value your experience and satisfaction with our product and services. To calibrate the system to a new user, please follow these instructions.

1. Place the **eye-tracker bar** in the **slot** at the bottom of the tablet. The magnetic strip will lock the eye-tracker in place. Make sure the USB connector on the end is connected to the USB port on the Surface Pro.
2. Locate the Tobii Experience icon on your screen or on the taskbar (black bar at the bottom). It will likely be at the top of your screen. The icon on the taskbar is shown in the yellow circle at the bottom of the pictured screen.
3. The icon at the top needs to be double-clicked, the taskbar only needs to be clicked once. Touch Click on either of the icons.
4. You should see the Tobii Experience screen, shown in the picture. At the top, near the VisuALS name is the icon to click (it looks like a gear), shown circled in red on the pictured screen. Touch Click on that.

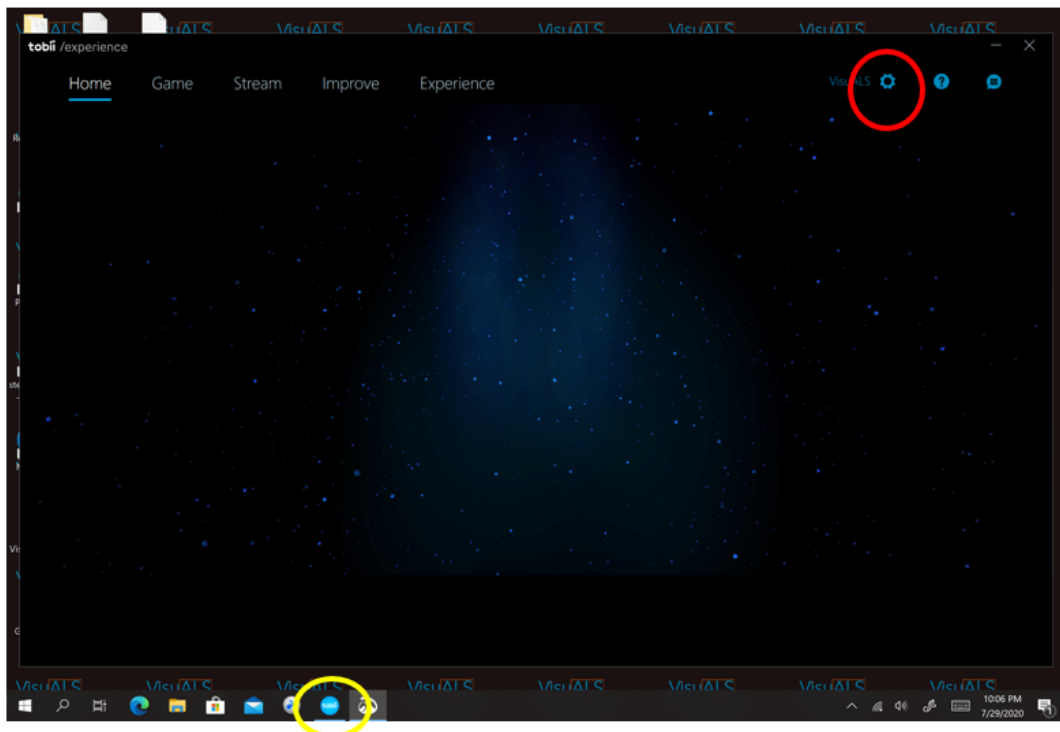

5. The Tobii Eye Tracker 5 Menu will open, shown in the picture below. The first item should be entitled “Improve my calibration”, shown in the red oval. Touch click on the blue button to start the calibration process.
6. For better accuracy, perform the calibration **twice** back-to-back

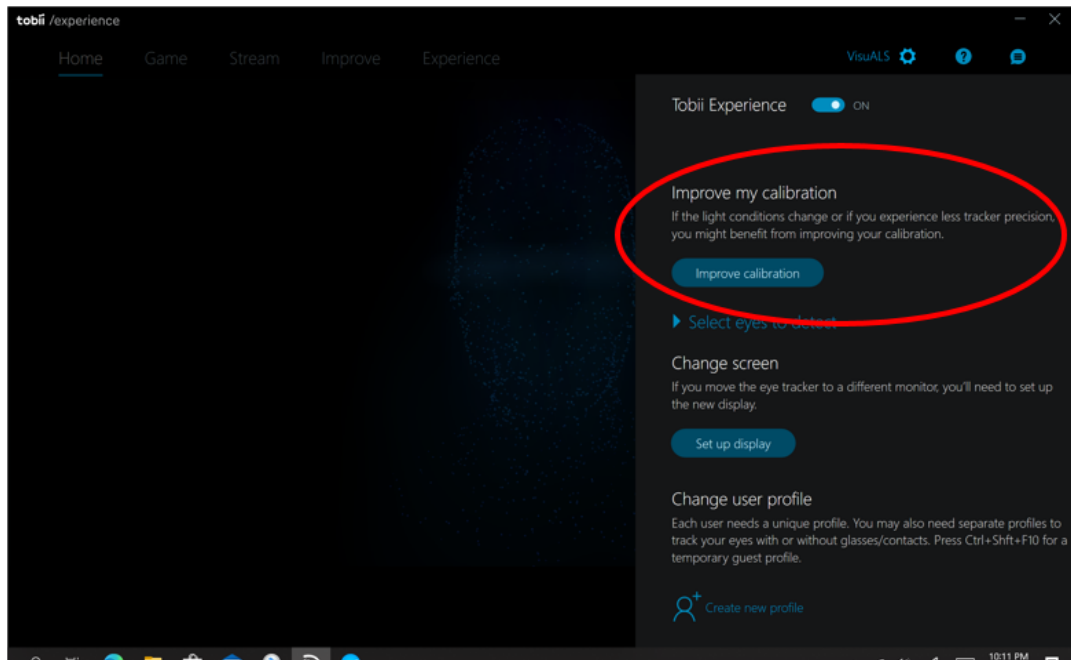

For more instructions and videos please visit <https://visuals.tech/video-tutorials/>
Or call our toll free number **(866) 767-4997**